




The F&P myMask™ App is designed to support patient mask setup. F&P myMask helps you and your patients effectively fit, fine-tune and clean F&P CPAP* masks for the treatment of obstructive sleep apnea.


Fitting Your Mask




1 Ensure the headgear clips are unhooked from the frame.
Hold the Front of the mask with one hand and the headgear with the other.

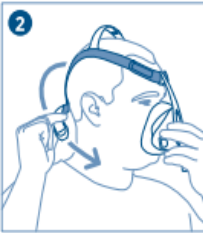


2 Place the silicone seal onto your face and guide the headgear over your head.

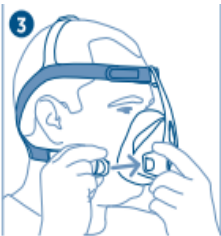


3 Hook the unattached Headgear clip onto the frame.

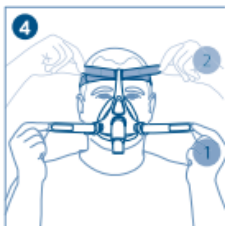


4 Gently tighten the headgear straps, starting the grey bottom straps, followed by loosely fitting blue forehead straps.

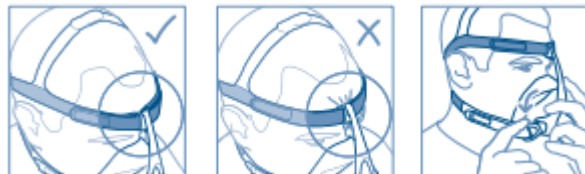


Assembling your Mask

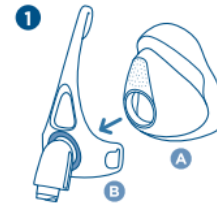

1 Push the Seal (A) firmly into place on the Frame (B).

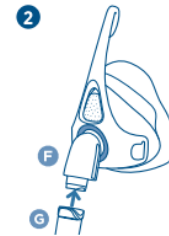

2 Push the Swivel (G) onto the Elbow (F).

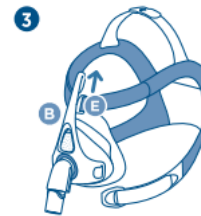

3 Slide the Forehead Clip (E) onto the top of the Frame (B).



4 Hook the Headgear Clips (D) onto the Frame (B).